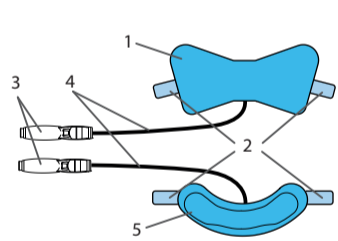
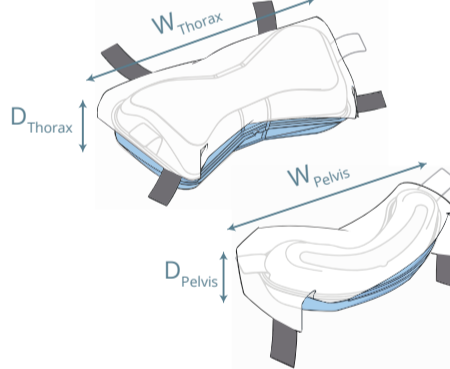
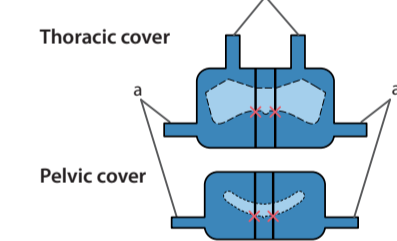
## IPV Inflatable Prone Ventilation

## COV (Sold Separately)

	W <sub>THORAX</sub>	D <sub>THORAX</sub>	W <sub>PELVIS</sub>	D <sub>PELVIS</sub>	Max
L	49 cm 19.3 in	12 cm 4.7 in	46 cm 18.1 in	10 cm 3.9 in	150 kg 330 lb
XL	53 cm 20.9 in	14 cm 5.5 in	51 cm 20.1 in	12 cm 4.7 in	180 kg 396 lb

### Instruction for Use (ENG) IPV (Inflatable Prone Ventilation)

**Description**  
The IPV is a single-patient use inflatable cushion, used in combination with the compatible Ergotrics' covers. Inflated and deflated with the Ergotrics' Hummingbird (HMB) using compressed medical air, the IPV lifts and positions the patient in prone position. For best result, use Ergotrics' IBO (Inflatable Board) for proning the patient.

#### The IPV consists of:

1. Thoracic cushion
2. Side tape
3. Valve system
4. Tube
5. Pelvic cushion

#### Compatible Covers (COV):

Catalogue number IPV	Catalogue number COV
IPV-VUL-xx	COV-PAU-xx, COV-PDU-xx
IPV-VUX-xx	

**Intended purpose**  
To provide abdominal unloading during prone ventilation.

**Indication for use**  
For use during prone ventilation.

#### Operating instructions

**Recommended use:** applying IPV in supine position.

1. Position the patient in supine position with arms spread (Fig. A).
2. Apply the covers on the patient using the skeleton as reference (Fig. B1). The line indicating the direction of tilting should be placed medially. The cross should be exactly on the xiphoid and pubic symphysis.
3. Attach the cover side tapes (a) tightly to the patient. Attach the side tape at the tilt side first.
4. Stick the IPV cushions in the middle of the covers (Fig. C1 & C2).
5. Diver the IPV tubes towards the umbilicus (Fig. C).
6. Attach the IPV side tapes tightly to the patient's body. Start with the tape at the tilt side.
7. Place the patient's arms along the body, with one arm over the tubes (Fig. D).
8. Attach the shoulder tapes (b) of the thorax cover tightly to the patient's body, diagonally across the patient's shoulder (Fig. D).
9. Cross the patient's ankle of the left side over the ankle of the tilt side (Fig. D).
10. Bring the patient in the prone position (Fig. E). For best result, use Ergotrics' IBO (Inflatable Board) for proning the patient.
11. Release all side tapes.
12. Inflate first the thoracic IPV cushion then the pelvic IPV cushion using the Ergotrics' HMB (Fig. H1). Stop inflation when you hear the whistle (Fig. F1) and close the pinch clamp (Fig. F2). **Remark:** Hold the IPV valve with one hand while pushing the HMB against it with the other hand.
13. Confirm correct positioning of the patient on the IPV. Reposition patient as required.
  - The thoracic cushions should not extend below the rib cage or above the clavicles.
  - The pelvic cushion should not extend over the groin.
  - The abdomen should be unrestricted.

**NOTE:** Use the HMB's horn to deflate the IPV cushions (Fig. H2) and release pressure if desired. Use the nose of the HMB to inflate the IPV if necessary (Fig. H1).

**Warnings**  
The IPV has been designed for single-patient use. Compatible Ergotrics covers and IPV side tapes should be exchanged after every use.  
Only inflate using the HMB with compressed medical air at 4-6bar/58-87psi. Higher pressures may cause damage.  
See table for maximum patient weight.  
Store under normal warehouse conditions.  
Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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